


Sung Kwang Meditech

**For surgical patient
& Rehabilitation
Radiology Products**

Operating Room

Snow-pac
Several device
Position pad
Gel product
Case
Silicon product

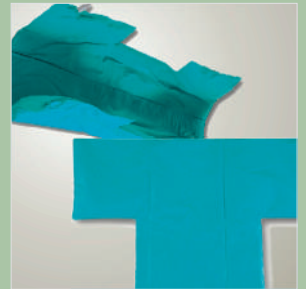


Rehabilitation

Gel wheelchair cushion pad
Gel wheelchair back cushion pad



Silver

Gel embo cushion mattress

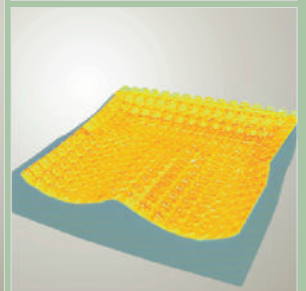

Radiation Protection For:
CardiacCathLab
InterventionalRadiology
Pain Management



Operating Room

SNOW PAC

FOR SURGERY POSITIONING PATIENTS

Snow Pac provides more stable and comfortable positioning for patient, saves valuable time for the entire O.R. team in every case.


It prevents pressure points.

aids to achieve comfortable position.

It aids faster, easier set-up.

aids better surgery position presentation.

PRINCIPLE OF SYSTEM

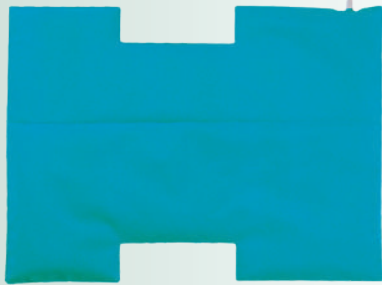

Before

Snow-Pacs are soft pads (available in various sizes and shapes) that are filled with thousands of tiny plastic beads.

The Snow-Pac is molded around the patient and then a suction line is attached to it.

As a vacuum is created inside the Snow-Pac within 15 seconds, outside atmospheric pressure forces the beads together so they cannot move.



After

When the suction line is disconnected, the Snow-Pac retains its firm, molded shape and holds the patient securely in position.

After surgery, the valve is opened, and as air reenters into the Snow-Pac it becomes soft again, ready for reuse.

EFFECT

More Stable

Because Snow Pacs are molded completely and firmly to shape of the patient's body and more securely than sandbags, towel rolls and other devices that are merely propped against the patient.

More Comfortable

The apparent wrinkles are inverted-drawn inward by suction and will not irritate the skin.

Better Presentation

With Snow Pacs the patient can be positioned exactly as the surgeon desires for the best presentation of the operative site. Snow Pacs conform to any shape, so the patient's position does not have to be compromised.

More Versatile

With Snow Pacs patient can often be placed in positions that could not be easily achieved using other positioning devices.

No Pressure Points

A Snow-Pac supports all parts of the patient's body uniformly, like a floatation mattress. Weight is distributed evenly, instead of being supported only by the bony prominences, which leads to pressure point injuries.

Saves Time

Patients can be positioned much more quickly and easily using Snow Pacs, and if necessary, be repositioned during a case.

OTHER FEATURES

Any Vacuum Source

Use wall suction, aspirator pump or any other convenient O.R. vacuum source.

Reusable

Snow Pacs give years of O.R. service. Rubberized vinyl fabric is soft but durable.

Radiolucent

Excellent radio transparency with only minimal and clearly identifiable artifacts.

Clean, Disinfect, Sterilize

Can be cleaned with regular soaps and liquid disinfectants or sterilized by gas.

Saves Money

Snow Pacs save the cost of sandbags, towel rolls and other positioning devices, and they save time in every case.

Repair Service

Factory inspection and repair service is available at nominal cost after warranty period.
Full One Year Warranty.

INDICATION

KIDNEY POSITION, LATERAL POSITION



A type



A-1 type
A-1-1 type



A-1-2 type
A-1-3 type

Easily up in nephrectomy (A type), and also for handicapped person with cerebral palsy.(A-1 type).



A type	73(L) * 100(W)cm
A-1 type	73(L) * 100(W)cm
A-1-1 type	90(L) * 100(W)cm
A-1-2 type	73(L) * 100(W)cm(U)
A-1-3 type	90(L) * 100(W)cm(U)

THYROID POSITION



B type
B-1 type

Provides hyperextension, prevents postoperative discomfort.

B type	72(L) * 88(W)cm
B-1 type	118(L) * 88(W)cm

HEAD STABILIZATION POSITION



C type

Stabilizes head in eye, facial or dental surgery.

C type	30(L) * 40(W)cm
--------	-----------------

SUPINE POSITION



D type




Supports and eliminates back pains during long procedures.

Prevents foot drop splint in Emergency.

D type	30(L) * 55(W)cm
--------	-----------------

VARIOUS FULL BODY POSITION

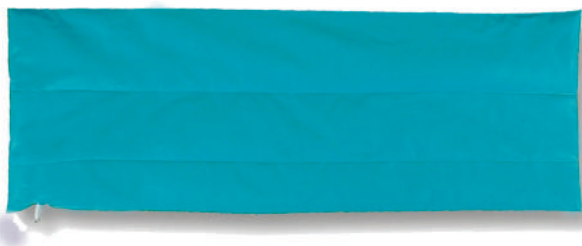

E type

E type	165(L) * 72(W)cm
--------	------------------

Operating Room

CASES

NEEDLE COUNTER & SHARPS COLLECTOR

inner : magnet
For surgery in OR
It aids to count needle thoroughly.




SK-18



SK-18-1

NEEDLE HUNTER

It finds needles by rolling on the floor

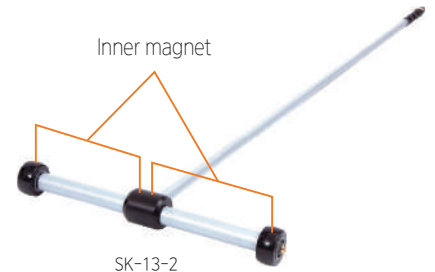

SK-13-2

(External)

SK-18 (S)	14(L) * 4(W) * 1.5(H)cm
SK-18-1 (L)	17(L) * 8(W) * 2(H)cm
SK-13-2	100(L) * 30(W)

MICRO INSTRUMENT ONE FLOOR CASE



SK-19-a (1F)



(External)

SK-19-a (1F)	26(L) * 17(W) * 2.5(H)cm
--------------	--------------------------


MICRO INSTRUMENT TWO FLOOR CASE



SK-19 (2F)



SK-19 (2F)

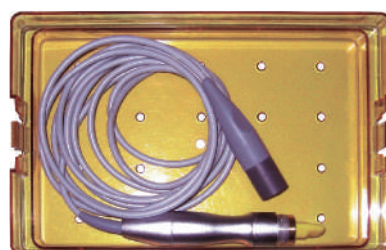

For microsurgical and other delicate instruments, for maximum protection and flexible handling, made of resilient polymer.

Protruded silicon embomat is placed on the bottom prevents instrument to move freely.

Cases are designed to withstand repeated exposure to 134°C steam temperature.

Steam / EO / Plasma Sterilizable

(External)

SK-19(2F)	26(L) * 17(W) * 4(H)cm
-----------	------------------------

MICRO INSTRUMENT ONE FLOOR CASE



SK-19-S

For microsurgical and other delicate instruments, for maximum protection and flexible handling, made of resilient polymer.

Protruded silicon embomat is placed on the bottom prevents instrument to move freely.

Cases are designed to withstand repeated exposure to 134°C steam temperature.



SK-19-1

Steam / EO / Plasma Sterilizable




SK-19-3

(External)


SK-19-S	16(L) * 6(W) * 3(H)cm
SK-19-1	25(L) * 5(W) * 2.5(H)cm
SK-19-3	30(L) * 10(W) * 2.5(H)cm

Operating Room

MICRO INSTRUMENT ONE FLOOR CASE

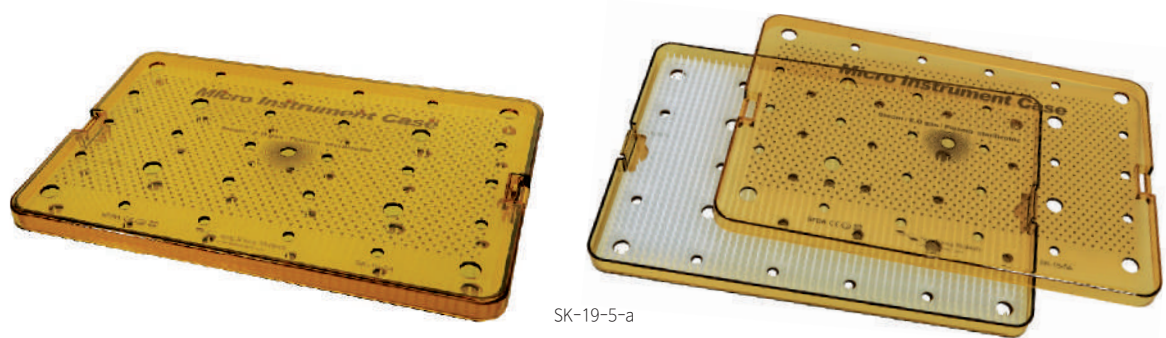

SK-19-5-a

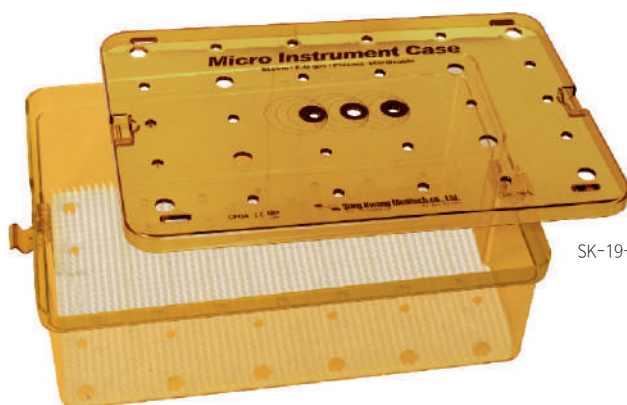

SK-19-5-b




Steam / EO / Plasma Sterilizable

(External)

SK-19-5-a(1F) 38(L) * 26(W) * 2.5(H)cm

SK-19-5-b(1F) 39(L) * 27(W) * 12(H)cm

MICRO INSTRUMENT TWO FLOOR CASE

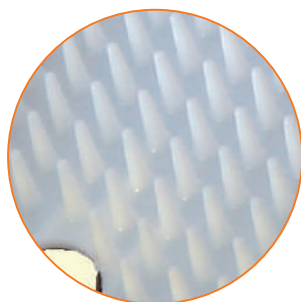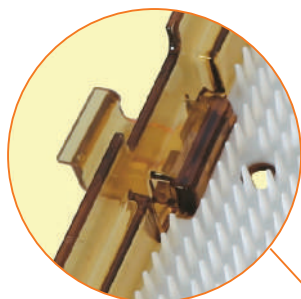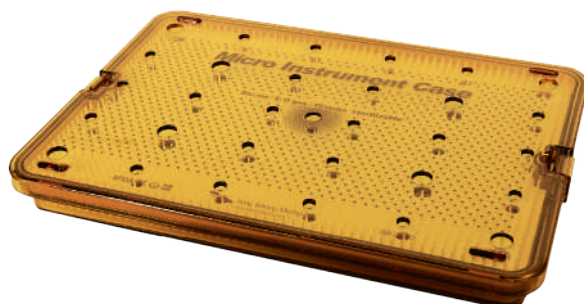

SK-19-5 (2F)

For microsurgical and other delicate instruments, for maximum protection and flexible handling, made of resilient polymer.

Protruded silicon embomat is placed on the bottom prevents instrument to move freely.


Cases are designed to withstand repeated exposure to 134°C steam temperature.

Steam / EO / Plasma Sterilizable

(External)

SK-19-5(2F) 39(L) * 27(W) * 4(H)cm

LAPAROSCOPE CASE

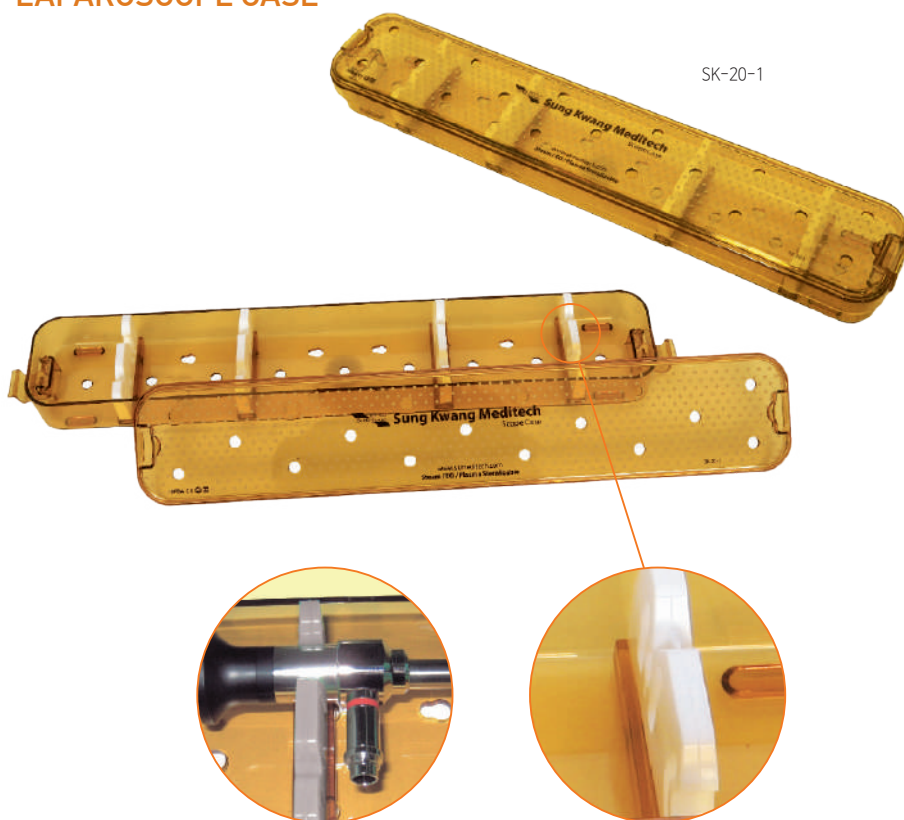

For Laparoscope.

Silicon bar is placed to prevent laparoscope to move freely.

Cases are designed to withstand repeated exposure to 134°C steam temperature.

Steam / EO / Plasma Sterilizable

(External)	
SK-20-1	48(L) * 9(W) * 5(H)cm

TELESCOPE CASE



For Telescope.

For Arthroscope Silicon bar is placed to prevent laparoscope to move freely.


Cases are designed to withstand repeated exposure to 134°C steam temperature.

Steam / EO / Plasma Sterilizable

(External)	
SK-20-2	28(L) * 7(W) * 5(H)cm

Operating Room

ENDOSCOPE INSTRUMENT CASE

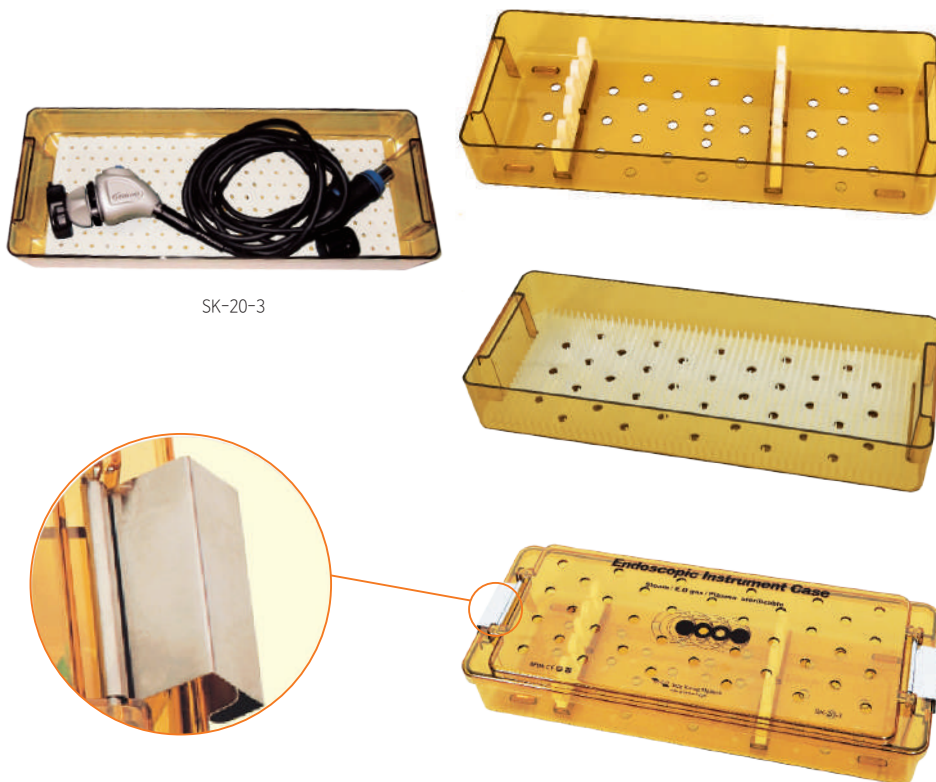

SK-20-3

For Endoscope Instrument.

Silicon bar is placed to prevent laparoscopic instrument to move freely.

Cases are designed to with stand repeated exposure to 134°C steam temperature.


SK-20-3 is used for endoscope camera case.

Steam / EO / Plasma Sterilizable

(External)

SK-20-3 37(L) * 15(W) * 7(H)cm

LAPAROSCOPIC INSTRUMENT CASE

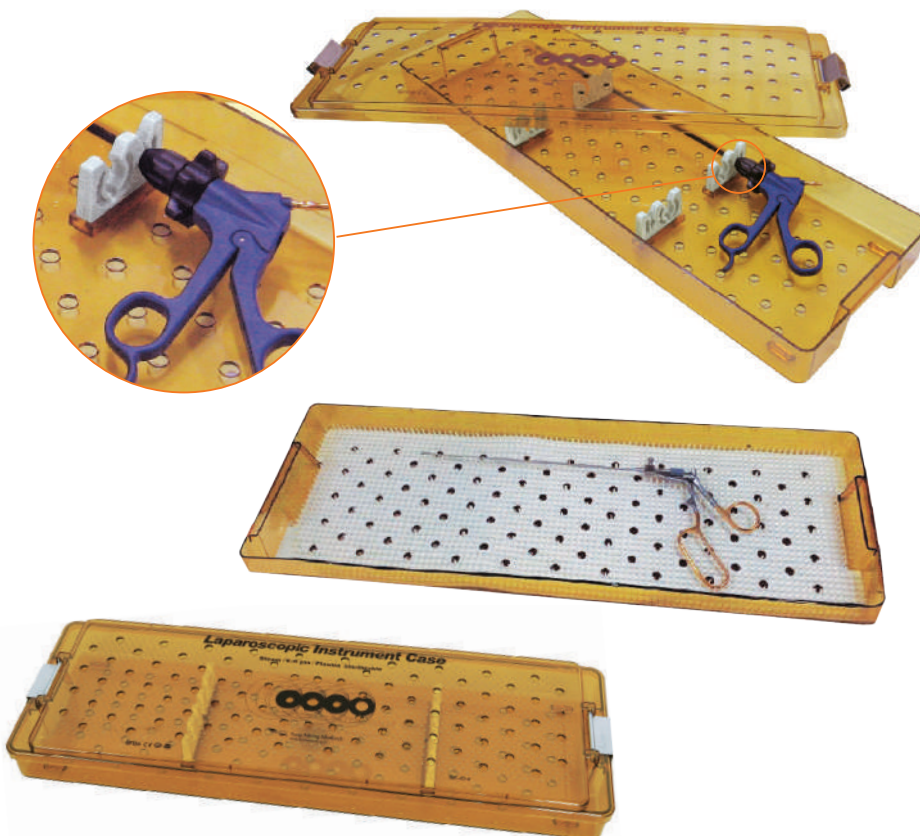

SK-20-4

Silicon bar is placed to prevent laparoscopic instrument to move freely.

Cases are designed to with stand repeated exposure to 134°C steam temperature.


Steam / EO / Plasma Sterilizable

It contains 2pcs or more of laparoscopic instrument.



(External)

SK-20-4 60(L) * 20(W) * 6(H)cm

LONG SCOPE INSTRUMENT CASE

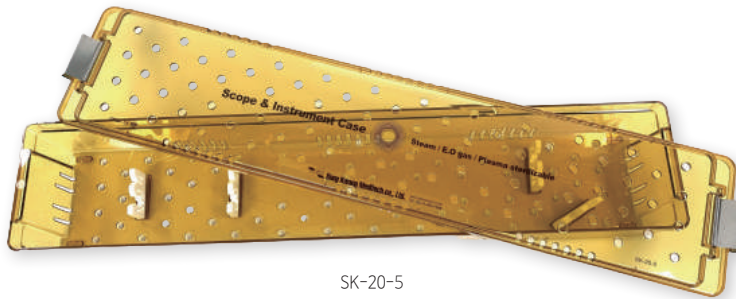

SK-20-5

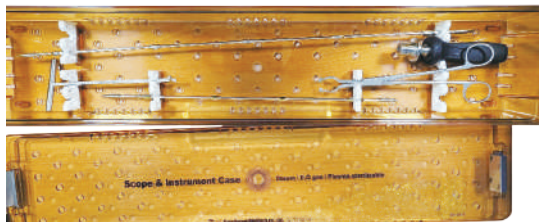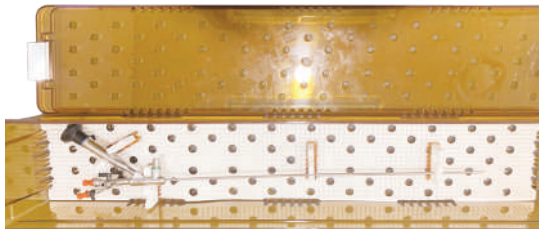

Sterilization cases used during laparoscopic long scope and robotic instrument surgery.

Silicon bar or mat is placed to prevent movement of instruments used in obstetrics, urology, otorhinolaryngology, surgery and robotic surgery.

Two or more laparoscopic instruments can be inserted.




Cases are designed to withstand repeated exposure to 134°C steam temperature.

Steam / EO / Plasma Sterilizable

(External)

SK-20-5 72(L) * 15(W) * 10(H)cm

LAPAROSCOPIC INSTRUMENT CASE

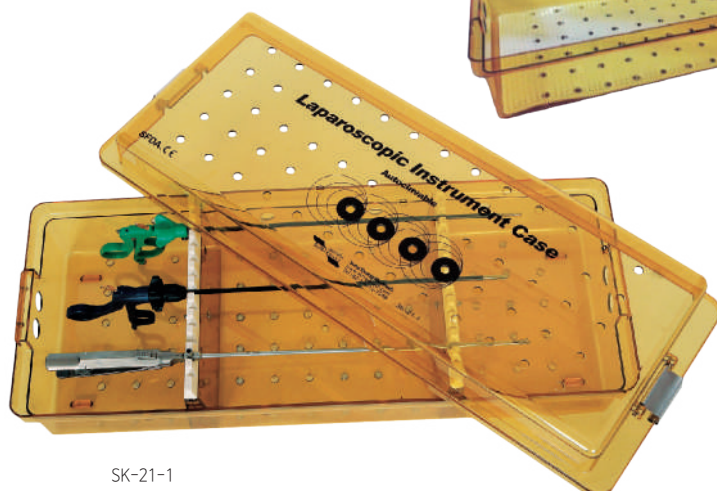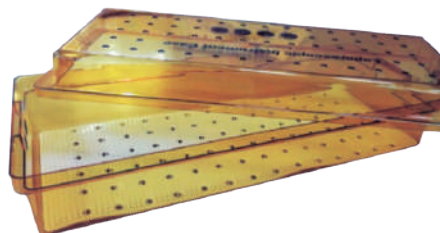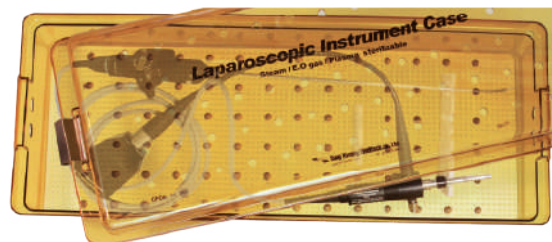

SK-21-1

Silicon bar is placed to prevent laparoscopic instrument to move freely.

Cases are designed to withstand repeated exposure to 134°C steam temperature.

Steam / EO / Plasma Sterilizable


It contains 10pcs of laparoscopic instrument.

(External)

SK-21-1 64(L) * 25(W) * 15.5(H)cm

Operating Room

OPHTHALMIC INSTRUMENT CASE

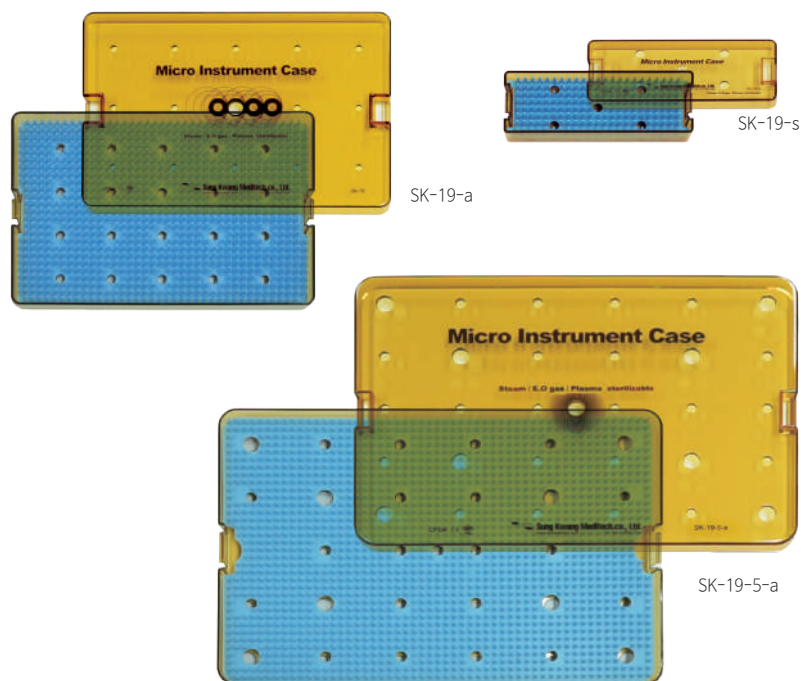

For microsurgical and other delicate instruments, for maximum protection and flexible handling, made of resilient polymer.

Protruded silicon embomat is placed on the bottom prevents instrument to move freely.

Cases are designed to withstand repeated exposure to 134°C steam temperature.


Steam / EO / Plasma Sterilizable

	(External)
SK-19-S	16(L) * 6(W) * 3(H)cm
SK-19-a (1F)	26(L) * 7(W) * 2.5(H)cm
SK-19-5-a (1F)	38(L) * 26(W) * 2.5(H)cm

BIOPSY CASE

R2	Delphain	Pyramidal	L2
R3			L3
R. Jungular	R. Thyroid	L. Thyroid	L. Jungular
R4			L4
R5	R.CN	L.CN	L5
R. Para. T			L. Para. T

SK-21-a-1 (for Thyroid)

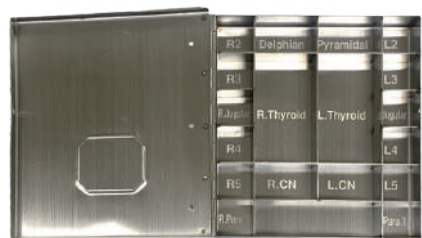

Rt Level 1		Rt Level 1	
Rt Level 2		Rt Level 2	
Rt Level 3		Rt Level 2	
Rt deep margin	Rt rotter's	Lt deep margin	Lt rotter's
Rt. axillary	Rt. sentinel	Lt. axillary	Lt. sentinel

SK-21-a-2 (for breast)



1	2	3	4	5	6	7	4sa		
8	9	10	11	11p	11d	12	14v	4sb	4d

SK21-a-3 (for stomach)



5	6	8	12a	12b	12c	12h	12p
13	14a	14v	16	16a2	16b1	17	

SK-21-a-4 (for biliary. Pancreas)



Material : stainless steel with marked L/N number,
Use : for specimen storage

	(External)
SK-21-a-1 for Thyroid	16.5(W) * 19(L) * 3.2(H)
SK-21-a-2 for breast	14.5(W) * 15(L) * 3.2(H)
SK-21-a-3 for stomach	31.5(W) * 8.5(L) * 3.2(H)
SK-21-a-4 for biliary. Pancreas	30(W) * 8.5(L) * 3.2(H)

POSITION PADS

PAD FOR SUPINE POSITION



Special sponge(inner)
Good positioner
Anti-bedsores effect Very soft, Good elasticity Good endure.
Poly urethane vinyl(outer)
Water resistant.

SUPINE

Head, foot pad	
SK-1	22(L) * 16(W) * 11 (5.5)(H)cm
Head pad	
SK-1-1	22.8(O.D) * 11.5(I.D) * 5(H)cm
Shoulder pad	
SK-2	23(L) * 14(W) * 7(H)cm


PAD FOR PRONE POSITION



PRONE

Chest pad (thick, thin)	
SK-5-1	18(L) * 52(W) * 13(H)cm
Head pad	
SK-6	32(L) * 25(W) * 10(H)cm
SK-6-1	24(L) * 26.5(W) * 18(H)cm
Knee pad	
SK-7	30(L) * 20(W) * 6(H)cm
Leg pad	
SK-3	40(L) * 17(W) * 17(H)cm

PAD FOR LATERAL POSITION

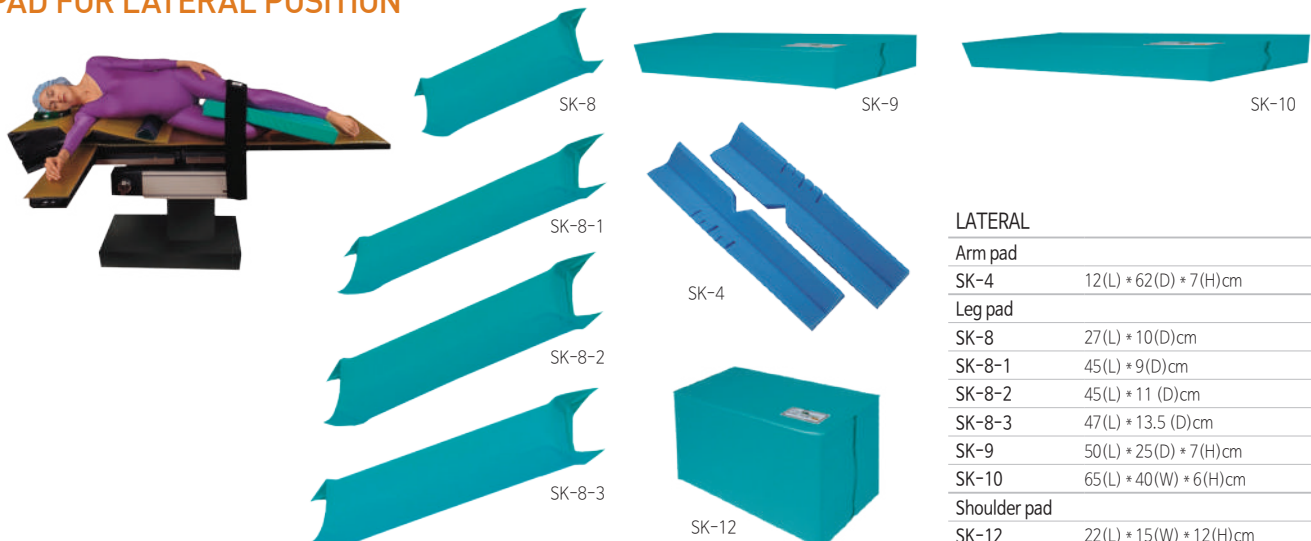

LATERAL

Arm pad	
SK-4	12(L) * 62(D) * 7(H)cm
Leg pad	
SK-8	27(L) * 10(D)cm
SK-8-1	45(L) * 9(D)cm
SK-8-2	45(L) * 11 (D)cm
SK-8-3	47(L) * 13.5 (D)cm
SK-9	50(L) * 25(D) * 7(H)cm
SK-10	65(L) * 40(W) * 6(H)cm
Shoulder pad	
SK-12	22(L) * 15(W) * 12(H)cm

Operating Room

GEL PRODUCTS

Capillary closure is at the root of ulcer formation.

The blockage or damage of capillaries prevents the natural maintenance of healthy skin.

In operation, the use of proper positioning devices prevent excessive pressure build-up and capillary closure.

Evenly distribute weight and provide constant support beneath patients.

Gel products are cost effective, latex free, fast & easy cleaning, radiolucent tissue equivalent.



SK-23



SK-24



SK-24-1



SK-25



SK-29



SK-22-1




SK-22-2



Pressure Injury Free Gel Products



SK-26

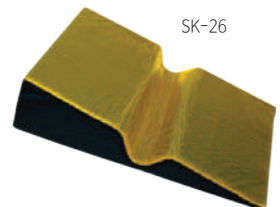

SK-27



SK-22



SK-29



SK-28



DOUGHNUT GEL HEAD PAD



Doughnut gel head pad provides excellent protection when the patient is positioned for surgery by supine position.

SK-22 (Adult)	20(D) * 4.5(H)cm
SK-22-3 (Child)	15(D) * 3.5(H)cm
SK-22-4 (Baby)	9.2(D) * 2.4(H)cm

C-DOUGHNUT GEL HEAD PAD



C-doughnut gel head pad provides clear air passage way when the patient is positioned for surgery by lateral or prone position.

SK-22-1 (Adult)	21(D) * 4.5(H)cm
SK-22-1-1 (Child)	15(D) * 3.5(H)cm
SK-22-1-2 (Baby)	9.2(D) * 2.4(H)cm

GEL OPHTHALMIC HEADREST



It was designed for stable head placement for all ophthalmic and oral or facial procedures.

Also provides patient with comfort while under local anesthesia, minimizing patient movement.

It was also designed for surgeon's functional arm support.

SK-22-a (Adult)	31(L) * 27(W) * 10(H)cm
SK-22-b (Child)	31(L) * 25(W) * 7.3(H)cm

GEL OPHTHALMIC CRADLE HEADREST




It was designed for ENT, plastic surgery, all ophthalmic cases.

It helps prevent risk of alopecia especially during long surgery.

Cradling of patient's head provides stability.

SK-22-c (Adult)	23(L) * 23(W) * 5.7(H)cm
SK-22-d (Child)	19(L) * 21.6(W) * 3.8(H)cm

GEL HEAD PAD

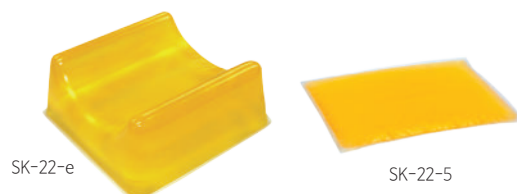

It provides optimum stability of patient's head while in the supine position.

Can also be used for position and protect the knee or calf.

SK-22-e (Adult)	18(L) * 18(W) * 8(H)cm
SK-22-5 (Baby)	16(L) * 12(W) * 1(H)cm

PRONE GEL HEAD PAD



It provides clear air passage way when the patient is positioned for surgery by prone position.

SK-22-2	27(L) * 24(W) * 15(H)cm
SK-22-2-1	27(L) * 21(W) * 8(H)cm

Operating Room


GEL BODY PAD



Gel body pad provides complete pressure reduction and shares across entire table surface

SK-23-2	84(L) * 50(W) * 1.3(H)cm
SK-23	160(L) * 50(W) * 1.3(H)cm
SK-23-1	183(L) * 50(W) * 1.3(H)cm

GEL BODY PAD 3-PIECE SET

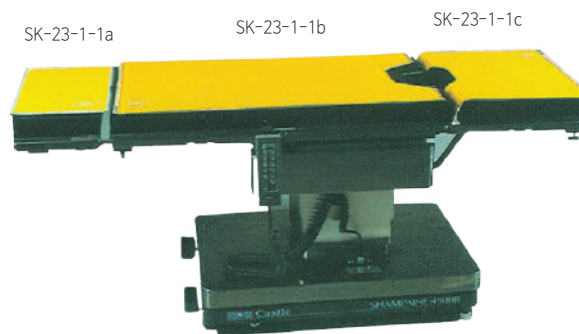

This set includes head, torso and foot section pads for use with segment tables.

SK-23-1-1 Gel body pad 3-piece set (a+b+c)	
SK-23-1-1a(Head)	50(L) * 23(W) * 1.3(H)cm
SK-23-1-1b(torso)	100(L) * 50(W) * 1.3(H)cm
SK-23-1-1c(foot)	50(L) * 50(W) * 1.3(H)cm


GEL INCUBATOR PAD



It provides gentle pressure for neo-natal patients in the NIC units. It is designed for the isolette beds.

SK-23-5	60(L) * 33(W) * 1.3(H)cm
SK-23-5-1	60(L) * 40(W) * 1.3(H)cm


GEL CHAIR PAD

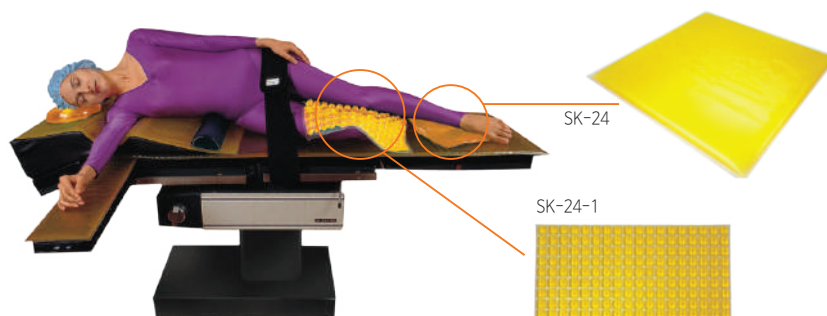

Gel chair pad is used underneath prominence area in a variety of applications.

SK-24	42(L) * 42(W) * 1.3(H)cm
SK-24-2	38(L) * 38(W) * 1.3(H)cm
SK-24-2-1	40(L) * 40(W) * 1(H)cm
SK-24-2-2	40(L) * 45(W) * 1(H)cm
SK-24-3	32(L) * 32(W) * 1.3(H)cm

GEL EMBO PAD

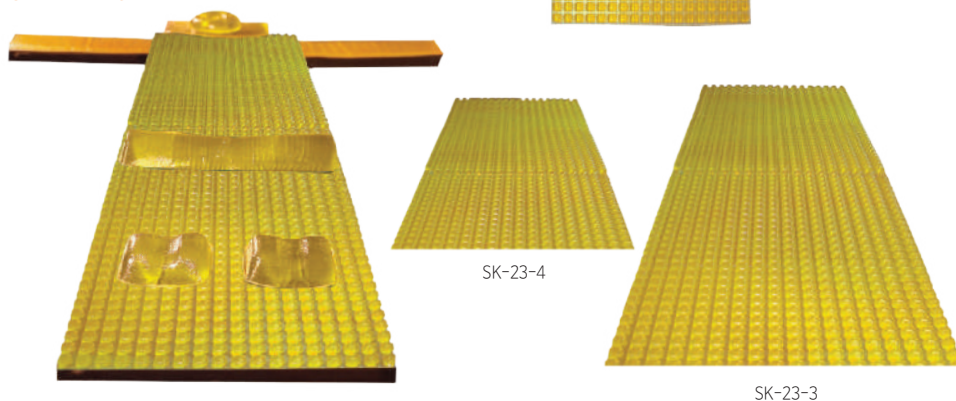

Gel embolus chair pad is used for iliac crest, under heels and elbows. Is used as kidney rest, axillary roll while patient in lateral position.

Gel embolus pad provides complete pressure reduction and shears cross entire table surface.

Gel embolus chair pad	
SK-24-1	40(L) * 45(W) * 1.6(H)cm
Gel embolus half body pad	
SK-23-4	80(L) * 45(W) * 1.6(H)cm
Gel embolus body pad	
SK-23-3	160(L) * 45(W) * 1.6(H)cm

GEL KNEE PAD




Gel knee pad is used for OBGY surgery in lithotomy position. It provides against nerve injury.

SK-25	40(L) * 24(W) * 1(H)cm
SK-25-1	50(L) * 52(W) * 1(H)cm

GEL LATERAL PAD/GEL



GEL THYROID PAD

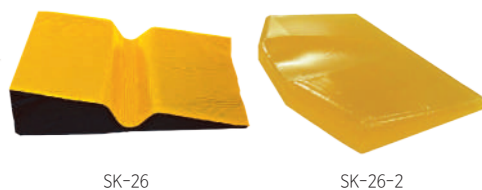

Gel lateral pad is designed to accommodate both the shoulder and provides exceptional pressure relief and nerve protection.

Gel thyroid pad is designed for thyroidectomy. It provides patient with gentle pressure while long surgery with hyperextension position. The tissue friendly feels make it very comfortable.

SK-26	70(L) * 50(W) * 12(H) / 0.8cm
SK-26-2	45(L) * 30(W) * 7(H)cm

GEL PATIENT POSITIONER



It is good for any procedure. versatile positioner.

SK-27-a(Small)	25(L) * 13(W) * 3(H)cm
SK-27-b(Large)	35(L) * 12(W) * 3(H)cm
SK-27-c(Extra long)	46(L) * 11(W) * 3(H)cm

GEL PRONE CHEST PAD



They were designed for uniformly conform to a patient s shoulder to chest area, with rounded edges for the axilla.

They provide optimum alleviating pressure on the thorax area.

SK-27-d(Right)	70/58(L) * 20(W) * 5(H)cm
SK-27-e(Left)	70/58(L) * 20(W) * 5(H)cm

GEL SHOULDER PAD (CHEST)



Semi-circular design provides maximum pressure protection while patient is in supine position.

SK-27, 27-4 can be used under the knee, the foot against the board.

While patient is in lateral position, SK-27 can be used for the shoulder protection.

In thyroidectomy , SK-27-4 is used for neck hyperextension.

while patient is in prone position SK-27-1, SK-27-2 can be used under the chest.

SK-27	47(L) * 10(W) * 5(H)cm
SK-27-1	47(L) * 15(W) * 8(H)cm
SK-27-2	47(L) * 15(W) * 12(H)cm
SK-27-3	25(L) * 15(W) * 8(H)cm
SK-27-4	52(L) * 10(W) * 8(H)cm

Operating Room

GEL HEEL PAD



Gel heel pad provides safety and protects heel area body prominence.

SK-28	20(L) * 11(W) * 7(H)cm
-------	------------------------

GEL HEEL SUPPORTER



GEL HEEL CUP




It protects heel area without hook and loop attachment.

It protects the patient's Achilles tendon area.

It supports heels. Appropriate for recovery.

SK-28-1	16(L) * 8.5(W) * 5(H)cm
SK-28-2	17(L) * 18(W) * 1.3(H)cm

GEL HEEL PROTECTOR

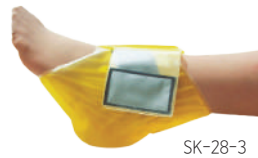

It supports heels appropriately for recovery.

SK-28-3	17(L) * 23(W) * 1.3(H)cm
---------	--------------------------

GEL ARMBOARD PAD



It is applied for supine position for protection of bony prominence area.

SK-29	55(L) * 15(W) * 1.3(H)cm
-------	--------------------------

GEL LATERAL ARMBOARD PAD



It is applied for lateral position. Designed to provide arm support and stability.

SK-29-1 (Small)	41(L) * 15(W) * 5(H)cm
SK-29-1a (Large)	55(L) * 15(W) * 6(H)cm

GEL WRIST AND ANKLE PROTECTOR



GEL ARMBOARD BINDER




It is used for nerve protection in fastening wrist and ankle.

SK-17-a Gel Wrist protector	26(L) * 6(W) * 1(H)cm
SK-17-b Gel Ankle protector	33(L) * 6(W) * 1(H)cm
SK-17-c Gel Armboard binder	21(L) * 6(W) * 1(H)cm

GEL WILSON FRAME COVER (2 PIECE SET)




GEL SK 5-1 COVER (2 PIECE SET)

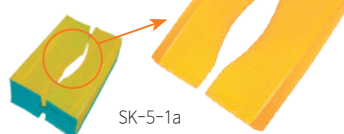

It provides comfort and distributes pressure in prone position spine surgery.

SK-17-1	75(L) * 24.5(W) * 1.3(H)cm
SK-5-1a	52(L) * 31(W) * 1.3(H)cm

GEL RELTON FRAME PAD (4 PIECE SET)

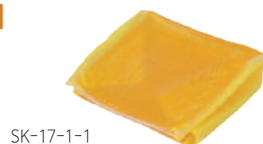

It provides comfort and safety in prone position spine surgery.

SK-17-1-1	1.3(thick)cm
-----------	--------------

GEL FOREARM PROTECTOR



SK-17-2



It is used for nerve protection of arm and elbow in supine position surgery.

SK-17-2 Gel forearm protector(L)

46(L) * 21(W) * 0.8(H)cm

SK-17-2-1 Gel forearm protector(S)

38(L) * 11.5(W) * 0.8(H)cm

GEL BODY BELT



SK-17-3



It helps safety, stability and tissue protection during surgery.

SK-17-3

50(L) * 8.5(W) * 0.8(H)cm

GEL SHOULDER PROTECTOR



SK-17-3-1

It is designed for patient in trendelenburg down position to protect shoulder areas.

SK-17-3-1

16(L) * 15(W) * 2(H)cm

GEL ALLEN PAD

SK-17-4-1



It is used lithotomy position for nerve protection.

SK-17-4-1

68(L) * 38(W) * 1(H)cm

GEL ANKLE HOLDER (2 PIECE SET)

SK-17-5



It is used in lithotomy position for nerve protection.

SK-17-5

40(L) * 3(W) * 1(H)cm

GEL SNOW PAC A-1 type COVER



SK-17-6



Designed for A-1 type snow pac. It provides comfort.

SK-17-6

100(L) * 73(W) * 1(H)cm

GEL SNOW PAC B type COVER



SK-17-6-1



Designed for B type snow pac.

SK-17-6-1

88(L) * 72(W) * 1(H)cm

GEL SNOW PAC C type COVER



SK-17-6-2



Designed for C type snow pac.

SK-17-6-2

30(L) * 40(W) * 1(H)cm


Operating Room

SILICON

SILICON ROD & MAT

Silicon rod & mat protect microinstruments in microsurgery.


It endures 300 degree C temperature up.

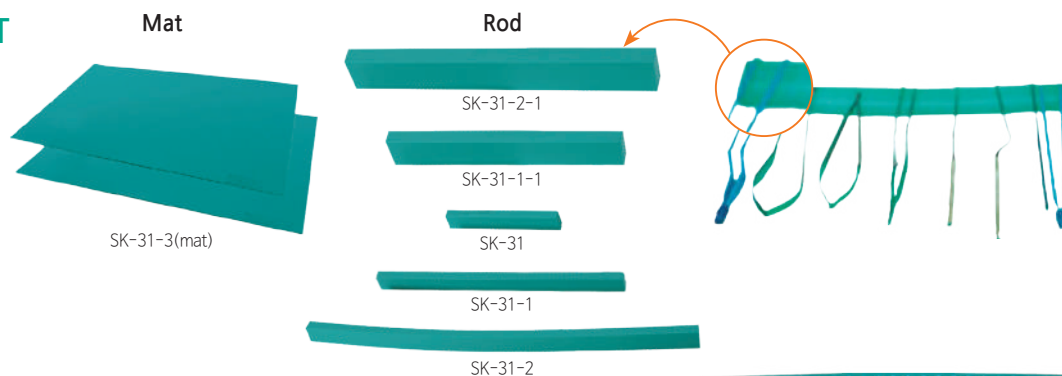


Hole mat



Embo mat

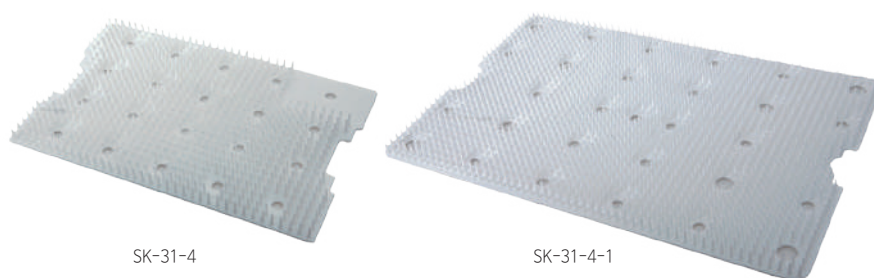

Silimagnet mat

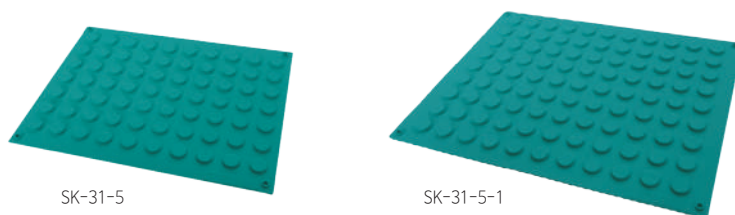

SILICON ROD

SK-31 (Rod)	1.5(L) * 10(W) * 0.8(H)cm
SK-31-1 (Rod)	1.5(L) * 20(W) * 0.8(H)cm
SK-31-1-1 (Rod)	1.5(L) * 20(W) * 2.8(H)cm
SK-31-2 (Rod)	1.5(L) * 30(W) * 0.8(H)cm
SK-31-2-1 (Rod)	1.5(L) * 30(W) * 2.8(H)cm

SILICON MAT

SK-31-3 (Mat)	50(L) * 33(W) * 0.2(H)cm
---------------	--------------------------

SILICON HOLE MAT

SK-31-3-a	50(L) * 25(W) * 0.2(H)cm
SK-31-3-b	33(L) * 25(W) * 0.2(H)cm
SK-31-3-c	25(L) * 25(W) * 0.2(H)cm
SK-31-3-d	50(L) * 33(W) * 0.2(H)cm

SILICON EMBO MAT

SK-31-4	23.5(L) * 15(W) * 1.5(H)cm
SK-31-4-1	37(L) * 24.3(W) * 1.5(H)cm

SILIMAGNET MAT

SK-31-5	38(L) * 30(W) * 0.6(H)cm
SK-31-5-1	40(L) * 35(W) * 0.6(H)cm

SILICON TUBE

SILI GAS LINE (SK-31-4-3)

In laparoscopic surgery used for CO₂ gas infusion.



SILI SUCTION TUBE (SK-31-4-4)

In neurosurgery, used for suction tube. It provides elasticity, flexibility



SK-31-4-3	5.0(I.D) * 7.0(O.D) mm
SK-31-4-4	6.5(I.D) * 11.0(O.D) mm

SNOW PAC



AIR SUCTION



It is used for lurgeric patient and unpositioned handicapped person. Make position easy and fast. Then, suction equipment is used.

RH B type	83(L) * 108(W)cm
HB-102a	33cm



RH B type

SEVERAL DEVICES

ARM PROTECTOR

SK-13



PROTECTIVE BODY BAND



SK-14

PROTECTIVE KNEE BAND



SK-14-1




SK-13	40.5(L) * 17(H) * 15.5(W)cm	Color : White
SK-13-2	100(L) * 30(W)cm	
SK-14	200(L) * 14(W)cm	
SK-14-1	70(L) * 7(W)cm	

GEL WHEELCHAIR CUSHION PAD WITH COVER


It was made with a high-quality polyurethane gel and PU foam.
It provides excellent pressure relief.
The gel material will not migrate, leak, evaporate, harden or dry out if the cushion is accidentally punctured.
The material is highly resistant to bacteria and bodily fluids, and can be easily cleaned with warm soapy water or a disinfectant.
It offers excellent pressure reduction and pain relief combination for people with low to moderate risk of skin breakdown.
It provides excellent immersion stability and pressure distribution.
It provides great comfort and excellent positioning stability. The gel material also helps dissipate heat buildup.
4" cushion provides long-term positioning support and excellent comfort.
It is applied for individuals with a history of skin breakdown or those who suffer from pressure sores.

GEL WHEELCHAIR CUSHION 2"

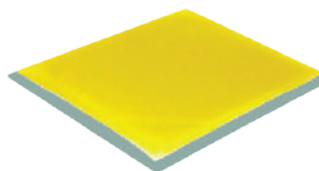


SK-1616-2	16(L) * 16(W) * 2(H) Inch
SK-1816-2	18(L) * 16(W) * 2(H) Inch
SK-1818-2	18(L) * 18(W) * 2(H) Inch

GEL WHEELCHAIR CUSHION 3"

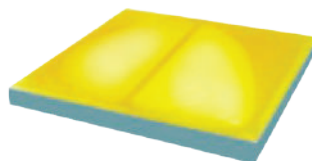


SK-1616-3	16(L) * 16(W) * 3(H) Inch
SK-1816-3	18(L) * 16(W) * 3(H) Inch
SK-1818-3	18(L) * 18(W) * 3(H) Inch

GEL WHEELCHAIR CUSHION CURVED 3.5"

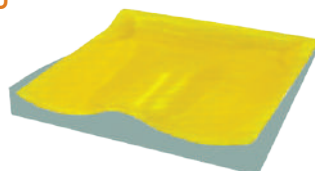

SK-C1616-3.5	16(L) * 16(W) * 3.5(H) Inch
SK-C1816-3.5	18(L) * 16(W) * 3.5(H) Inch
SK-C1818-3.5	18(L) * 18(W) * 3.5(H) Inch

GEL WHEELCHAIR CUSHION MORE CURVED 4"

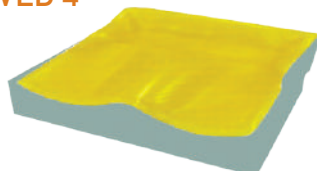

SK-MC1616-4	16(L) * 16(W) * 4(H) Inch
SK-MC1816-4	18(L) * 16(W) * 4(H) Inch
SK-MC1818-4	18(L) * 18(W) * 4(H) Inch


WHEELCHAIR CUSHION COVER



Bottom cover is made by non slip material.
Side cover is made by net shaped material.
Top cover is made by very soft, skinlike material.


SK-W-cover

GEL WHEELCHAIR EMBO CUSHION 2"

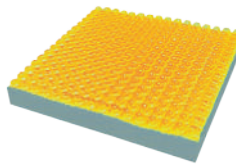


SK-E1616-2	16(L) * 16(W) * 2(H) Inch
SK-E1816-2	18(L) * 16(W) * 2(H) Inch
SK-E1818-2	18(L) * 18(W) * 2(H) Inch

GEL WHEELCHAIR EMBO CUSHION 3"

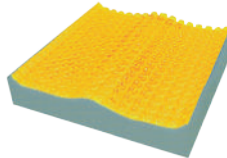


SK-E1616-3	16(L) * 16(W) * 3(H) Inch
SK-E1816-3	18(L) * 16(W) * 3(H) Inch
SK-E1818-3	18(L) * 18(W) * 3(H) Inch

GEL WHEELCHAIR EMBO CURVED CUSHION 3.5"

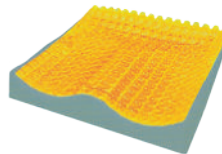

SK-EC1616-3.5	16(L) * 16(W) * 3.5(H) Inch
SK-EC1816-3.5	18(L) * 16(W) * 3.5(H) Inch
SK-EC1818-3.5	18(L) * 18(W) * 3.5(H) Inch

GEL WHEELCHAIR EMBO MORE CURVED CUSHION 4"

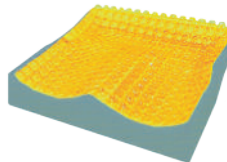

SK-EMC1616-4	16(L) * 16(W) * 4(H) Inch
SK-EMC1816-4	18(L) * 16(W) * 4(H) Inch
SK-EMC1818-4	18(L) * 18(W) * 4(H) Inch

GEL PRODUCTS-ACCESSORY

EMBO GEL WHEELCHAIR ARMBBOARD PAD



Adhesive tape attached in bottom

RH-3	23(L) * 7(W) * 1.6(H) cm
------	--------------------------

GEL TOILET COVER



It is very effective for users with existing skin breakdown.

RH-4(Dual)+cover	39(L) * 10(W) * 1(H)cm
------------------	------------------------

GEL COMMODE SEAT




RH-6 (close type)	45(L) * 45(W)cm
RH-6-1 (open type)	45(L) * 46(W)cm

Rehabilitation

GEL WHEELCHAIR BACK CUSHIONS WITH COVER

Polyurethane gel and PU foam helps comfort and durability for long-term.
The embossed gel feature aids comfort for long-lasting back and lumbar support in long-term seating environments.

EMBO GEL BACK CUSHION(A type)

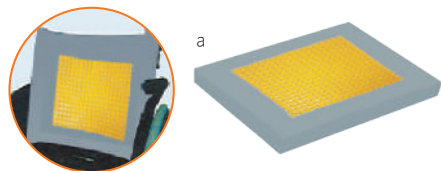

WHEELCHAIR BACK CUSHION COVER


SK-WB-cover




SK-B1417-a	14(L) * 17(W) * 2(H) Inch
SK-B1617-a	16(L) * 17(W) * 2(H) Inch
SK-B1817-a	18(L) * 17(W) * 2(H) Inch
SK-B2017-a	20(L) * 17(W) * 2(H) Inch

SK-B1417-b	14(L) * 17(W) * 2(H) Inch
SK-B1617-b	16(L) * 17(W) * 2(H) Inch
SK-B1817-b	18(L) * 17(W) * 2(H) Inch
SK-B2017-b	20(L) * 17(W) * 2(H) Inch

EMBO GEL BACK CUSHION (B type)

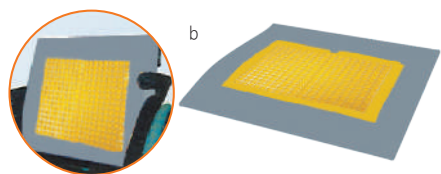


EMBO GEL BACK CUSHION(C type)

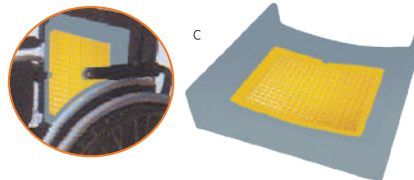

SK-B1417-c	14(L) * 17(W) * 2(H) Inch
SK-B1617-c	16(L) * 17(W) * 2(H) Inch
SK-B1817-c	18(L) * 17(W) * 2(H) Inch
SK-B2017-c	20(L) * 17(W) * 2(H) Inch

GEL HORSE S EQUESTRIAN PAD

RECTANGULAR ROUND

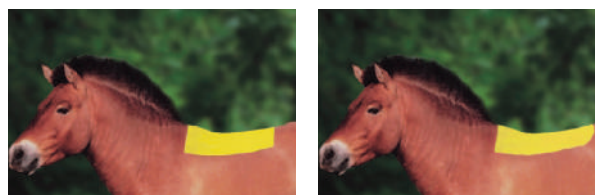

SK-EQ-a Rectangular shape with cover	36(L) * 50(W) * 1(H) cm
SK-EQ-a cover Rectangular shape cover only	36(L) * 50(W) * 1(H) cm
SK-EQ-b Round shape with cover	40(L) * 55(W) * 1(H) cm
SK-EQ-b cover Round shape cover only	40(L) * 55(W) * 1(H) cm

SILVER PRODUCTS


GEL EMBO CUSHION PILLOW / MATTRESS

Embo gel cushion pillow with cover (memory foam+embo gel)

CHARACTERISTICS :

Cspine supporting effect is developed by medical basis.
Softly, as humanengineering structure, it supports head.
After deep sleeping, it causes comfort.
As urethane foam materials made by physical method, although it is used for a long time, it doesn't transform.

Outer : 100% cotton velvet with silvercholid coating
Inner : cotton 75% polyester 25%
Materials : slowly recovery polyurethane foam 100% + embo gel pad
Color : silver, ivory
Kind : therapeutic, antibacterial, deodorizational.

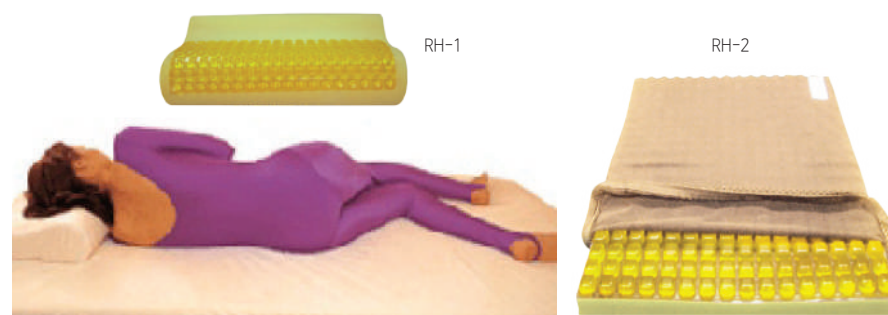

RH-1	50(L) * 33(W)cm
RH-2 (twin)	RH-2-1 (full)
RH-2-3 (queen)	RH-2-4 (king)

Cover Only

RH-2-cover	RH-2-1-cover
RH-2-3-cover	RH-2-4-cover

SEVERAL KINDS OF X-RAY APRON FOR SURGICAL PATIENT



SK-15 (Buckle type)

Personal protection in the medical use of radiation.

Modern radiology-supported diagnostic and therapeutic procedures extend the wear time for protective clot hing.

light weight, high quality, cost effective.

All apron items are available.

inner : natural rubber-based lead rubber flexible wearing comfort and best.

outer : nylon fabric, etc satisfies the workmanship hygienic requirements and the demands of medical use.

Pb equivalent : 0.25mm

Pb equivalent : 0.35mm

Pb equivalent : 0.50mm



SK-15-1 (Coat type)

MALE

	S		M		L		XL	
	L	W	L	W	L	W	L	W
Buckle type	93	58	98	60	103	63	108	66
Coat type (Front/back)	96/86	53	99/89	55	105/95	57	106/96	60
X-band type	94	55	96	57	100	60	104	62
Vest/Skirt type	59/50	49/42	60/52	51/43	61/53	53/46	63/54	56/49
panorama type	82	47	85	50	88	53	91	55

FEMALE

	S		M		L		XL	
	L	W	L	W	L	W	L	W
Buckle type	84	50	89	52	95	56	99	58
Coat type (Front/back)	85/75	47	91/81	49	95/85	52	100/90	54
X-band type	85	51	90	53	96	56	100	60
Vest/Skirt type	55/46	46/38	56/48	48/40	57/49	51/43	59/50	54/46
panorama type	76	43	81	47	84	50	87	53

± 2~4cm, unit : cm
W : width, L : Length



SK-15-4 (X-band type)



SK-15-5 (Buckle X-band type)

DROP OFF



Remove your apron quickly and easily with shoulder valcro.

Available to

SK-15, SK-15-1, SK-15-4, SK-15-5

Lead APRONS

SEVERAL KINDS OF X-RAY APRON FOR SURGICAL PATIENT



SK-15-2 (Vest & Skirt type)



SK-15-3 (Half type)



SK-15-3-1 (Panorama type)



SK-15-3-2

Thyroid cover protects thyroid gland in the medical use of radiation.
And disposable cover is also helpful in cleanliness.
All apron items are available.

inner : natural rubber-based lead rubber flexible wearing comfort and best
outer: nylon fabric, etc satisfies the workmanship hygienic requirements and the demands of medical use.

Pb equivalent : 0.25 mm
Pb equivalent : 0.35 mm
Pb equivalent : 0.50 mm

Color type



Blue Cross



Dark Blue



Pink




SK-16



SK-16-a



SK-16-b

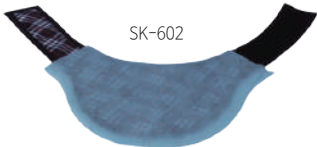

SK-602



SK-601



SK-500



SK-500-a



SK-16-4



SK-16-4-a



SK-16-5



SK-16-5-a

Like the Page Cap, our Regular Apron Cap is comfortable and lightweight.
An adjustable Velcro tab allows custom fitting.
Available in any apron fabric.
Please specify when ordering.
Order sizes :Male, Female
Protection : 0.25mm front
Closures : Velcro

Lead cap	SK-16-4
Lead cap	SK-16-4-a
Lead gonad shield	SK-16-5
Lead gonad shield	SK-16-5-a

SEVERAL KINDS OF X-RAY APRON FOR SURGICAL PATIENT



SKN-15 (Buckle type)



SKN-15-1 (Coat type)



SKN-15-4 (X-band type)



SKN-15-5 (Buckle X-band type)

Personal protection in the medical use of radiation.

Modern radiology-supported diagnostic and therapeutic procedures extend the wear time for protective clothing.

light weight, high quality, cost effective.

All apron items are available.

inner : lead free metal composites
flexible wearing comfort and best.

outer: nylon fabric, etc satisfies the workmanship hygienic requirements and the demands of medical use.

Pb equivalent : 0.25mm

Pb equivalent : 0.35mm

Pb equivalent : 0.50mm

DROP OFF



Remove your apron quickly and easily with shoulder velcro.

Available to

SKN-15, SKN-15-1, SKN-15-4, SKN-15-5

Non Lead APRONS

SEVERAL KINDS OF X-RAY APRON FOR SURGICAL PATIENT



SKN-15-2 (Vest & Skirt type)



SKN-15-3 (Half type)

SKN-15-3-1 (Panorama type)

SKN-15-3-2



All apron items are available,
 inner : lead free metal composites
 flexible wearing comfort and best
 outer : nylon fabric, etc satisfies
 the workmanship hygienic
 requirements and the
 demands of medical use.

Pb equivalent : 0.25 mm
 Pb equivalent : 0.35 mm
 Pb equivalent : 0.50 mm

Thyroid cover protects thyroid gland
 in the medical use of radiation.

Color type



Like the Page Cap, our Regular Apron
 Cap is comfortable and lightweight.
 An adjustable Velcro tab allows
 custom fitting.
 Available in any apron fabric.
 Please specify when ordering.


Order sizes :Male, Female
 Protection : 0.25mm front
 Closures : Velcro

Non Lead cap	SKN-16-4
Non Lead cap	SKN-16-4-a
Non Lead gonad shield	SKN-16-5
Non Lead gonad shield	SKN-16-5-a

LEAD-APRON HOLDER



LEAD-APRON WALL CONSOLE

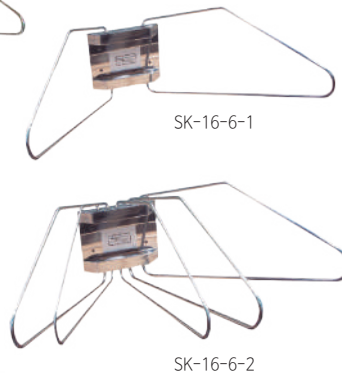

Apron holder will accommodate up to 2-10 aprons.

Unit is ideal if wall space is limited.

It is designed to prevent declining on side in moving.

Solid stainless steel construction prevents rust in long-term use.

High durability under heavy use.

Can be assembled after delivering.

Lead curtain provides separated radiation shielded space by simply drawing.

- protection : 0.5 mmpb
- Order made

FOR PREGNANT WOMAN BARRIER



TABLE LEAD CURTAIN



- Protection for pregnant woman barrier : 0.5 mmpb
- Order made

SK-16-6	71(L) * 63(W) * 140(H) cm
SK-16-6-a	50(L) * 63(W) * 140(H) cm
SK-16-6-1	25(L) * 23(W) cm
SK-16-6-2	25(L) * 23(W) cm
SK-16-7-1	57(W) * 60(L)/83(H) to 146(H) cm
SK-16-7-3	200(W) * 150(H) cm
SK-16-7-4	100(L) * 65(W) * 70(H) cm

LEAD BARRIER



Lead-glass viewing panel provides exceptional visibility.

- Protection : 1.76 mmpb
- Order made

SK-16-7 lead glass barrier	180(L) * 90(W) cm
Window Size:	80(L) * 90(W) cm
SK-16-7-a lead barrier	180(L) * 90(W) cm
SK-16-7-b	180(L) * 90(W) cm
Window Size:	20(L) * 30(W) cm
SK-16-7-c	180(L) * 90(W) cm
Window Size:	180(L) * 90(W) cm

99 ALUMILITE



99AL

Manufactured of lightweight nylon, the 99 Ultralite was designed to be comfortable and provide protection for the entire eye area without the need of an added side shield for lateral exposure.

The curved front and temples with rubber tips assures a close secure fit that also provides some splash protection.

An adjustable snap tip cord is standard with each model.

- ◆ Weight : 70 gram
- ◆ Lead Equiv Front : 0.75mm (+/-0.05) Lead glass
- ◆ Frame Material : Durable Nylon
- ◆ Available In : Plano (non-prescription), single Rx (low power), Bifocal Rx (low power)
- ◆ AVAILABLE COLORS : Silver.

ALUMILITE

99AL

70S ASTRO FLEX with SIDE SHIELDS



70s



A panoramic goggle constructed of impact resistant propionate.

The frame front has molded side shielding and is fitted to the brow for increased splash protection.

The temple length and frame angle are adjustable for a uniform custom fit.

Model 70S comes with Lead side shields for necessary protection from oblique angle exposure.

- ◆ Weight : 80 grams
- ◆ Lead Equiv Front : 0.75mm (+/-0.05) Lead glass
- ◆ Lead Equiv Side Shields : 0.50mm Lead sheet
- ◆ Frame Material : Propionate ◆ Color : Clear/Blue
- ◆ Available In : Plano (non-prescription), Single Rx, Bifocal Rx, Trifocal Rx, and Progressive.

ASTRO FLEX

70s

RE FITGUARD



Cocoon's Flexfit fitover is a versatile, high quality nylon frame. Featuring Flexfit technology, the fitover's temples can be benched (adjusted) easily for a secure, comfortable fit.

The large sized model is ideal for fitting over standard sized prescription eyewear with square or rounded lenses.

Blacked out side shields provide a streamlined look as well as 0.50mm lead equivalent protection.


- ◆ CE Marked : Yes
- ◆ Front Lens : 0.75mm Lead Glass ◆ Side Shields : 0.50mm pb Strips
- ◆ Available Prescription : None ◆ Prescription Range : None
- ◆ Weight : 108g ◆ Frame Size : 64-18-150
- ◆ Color : Black

FITGUARD

RE FITGUARD

Lead Masks

PANORAMA SHIELD

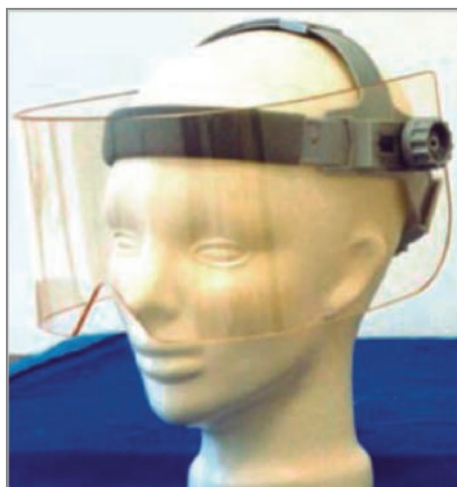

400PSM

The 400PSM Panorama Shield consists of a Face Formed Radiation Protective Lead Impregnated Acrylic Shield that is suspended from a dual adjustable headpiece.

The acrylic shield can be positioned at any angle to the face so as to minimize scatter radiation from passing under the shield by side adjustable dials. The headgear is adjustable in two positions.

An adjustable headband that saddles the crown of the head while connecting at a 90 angle to an adjustable headband that surrounds the head circumference from brow to back. The circumference portion of the headband has an adjustable dial in the rear to increase or decrease the hat size for a comfortable, snug fit.

Wipe clean with a soft cloth. Abrasive materials may scratch front lens.

Cannot be sterilized with high temperatures or high pressure.

◆ Weight : 350 grams

◆ Protection : .10mm Lead Acrylic ◆ Material : Acrylic/Nylon

PANORAMA SHIELD

400PSM

PANORAMA MASK



450FPM

The 450FPM Panorama Mask consists of a Face Formed Radiation Protective Lead Impregnated Acrylic Shield that is suspended from an adjustable headpiece.

The stationary acrylic shield extends from a headpiece brow connector for a length of 8 inches from top to bottom in front of the entire face. A support strap saddles the crown of the head while connecting at a 90 angle to an adjustable headband that surrounds the head circumference from brow to back.

The circumference portion of the headband has an adjustable strap in the rear to increase or decrease the hat size for a comfortable, snug fit. Wipe clean with a soft cloth.

Abrasive materials may scratch front lens. Cannot be sterilized with high temperatures or high pressure.


◆ Weight: 450 grams

◆ Protection: .10mm Lead Acrylic ◆ Material: Acrylic/Nylon

PANORAMA MASK

450FPM

BREAST SHIELD

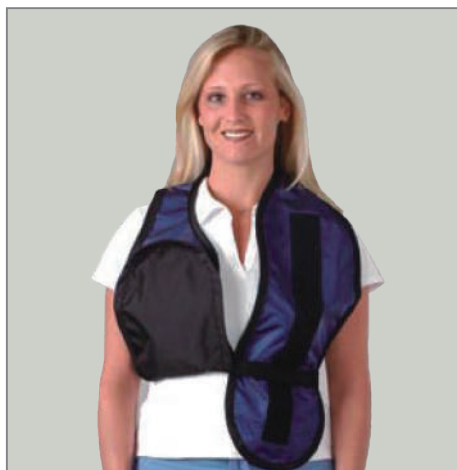

For use in spinal imaging. Shield features Velcro panels and adjustable belt for positioning.

Neck is lead free for complete spine exposure.

Great for scoliosis studies.

◆ Protection: .50mm

◆ Closures: Buckle

BREAST SHIELD

APSTOLE

- Lead Free** Our lightest 0.5mm lead-free material
- ◆ Environmentally Safe
 - ◆ Equivalent to 0.50mmpb at 100kVp

VEST-SKIRT COMBINATION



This Vest-Skirt combination is the preferred choice for complete protection and weight distribution.

The two piece design shifts 70% of the weight directly to the hips, reducing back fatigue.

Skirt features no-slip, quick release belt for added support.

Double padded shoulders for maximum comfort.

Chest pocket standard.



(Pictured with Thyroid Tether.)

- ◆ Protection : .50mm front / .25mm back
- ◆ Closures : Velcro and Quick-Release Buckle

VEST-SKIRT COMBINATION

APVS

WRAP AROUND BACK RELIEF

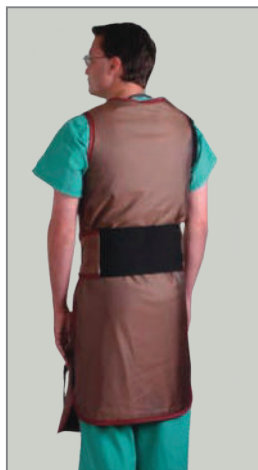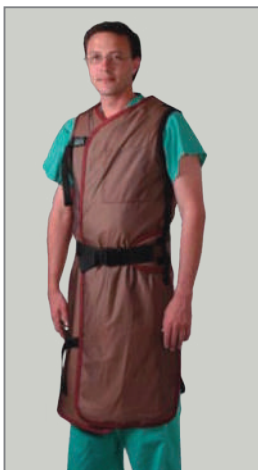

Offering full protection, this Back Relief design transfers the weight from the shoulders to the hips with the aid of a 6" bonded elastic belt.

(See features of our Back Relief belt.)

This apron features Velcro and buckle closures, as well as a Velcro Breakaway side seam for freedom of movement.

Double padded shoulders for added comfort and a chest pocket are standard features.

- ◆ Protection : .50mm front / .25mm back
- ◆ Closures : Velcro and Quick-Release Buckle

WRAP AROUND BACK RELIEF

APWABR

BACK RELIEF



Designed to transfer the weight from the shoulders to the hips.

The apron features a unique padded belt system constructed with a bonded elasticized panel.

The stretch proof panel will not lose its shape while providing increased comfort and back support through the life of the apron.

Extra thick shoulder pads, chest pocket and shoulder strap are standard. Optional tether on shoulder strap available upon request.

- ◆ Protection : .50mm front
- ◆ Closures : Velcro and Quick Release Buckle

BACK RELIEF APBR

APBR

FLEXBACK



Our Flexback apron is one of the most comfortable models you'll ever wear!

Criss-crossing back panels lift the weight off the shoulders and support the lower back.

Optional shoulder strap available upon request.

Extra thick shoulder pads are standard.

Chest pocket standard.

Shown with Thyroid Collar.

◆ Protection : .50mm front

◆ Closures : Velcro

FLEXBACK

APFB

VEST-SKIRT FLEXBACK



This design combines the benefits of a 2-piece apron with the support of our elasticized flexback apron.

You control the amount of back support by adjusting the inner elasticized quick release belt.

Chest pocket standard. Shown with Page Cap.

◆ Protection : .50mm front / .25mm back

◆ Closures : Velcro and Quick - Release Buckle

VEST-SKIRT FLEXBACK

APVSFB

BUCKLE APRON



Quick Release Buckle Apron is user friendly with large support straps criss-crossing the back allowing proper placement.

Belt is adjustable in order to accommodate varying sizes. Chest pocket and shoulder pads are standard.

Optional shoulder strap (as shown on Back Relief APBR above) available upon request.

◆ Protection : .50mm front

◆ Closures : Quick Release Buckle

BUCKLE APRON

APBA

FLEXBACK w/SURGICAL DROP SHOULDER



Two great aprons in one! This is the perfect choice for those who want the convenience of a Surgical Drop Apron and the comfort of our Flexback Apron.

Crisscrossing back panels transfer the weight from the shoulders and distribute it to the hips.

Lift the Velcro tabs for quick removal!

Extra thick shoulder pads and chest pocket are standard. Shown with Apron Cap.

◆ Protection : .50mm front

◆ Closures : Velcro

FLEXBACK w/SURGICAL DROP SHOULDER

APFBSB

Non Lead APRONS

SURGICAL DROP-OFF



Fluoro done Remove your apron quickly and easily without breaking the sterile field.

Extra thick shoulder pads for added comfort.

Chest pocket and hanging straps are standard.

◆ Protection : .50mm front

◆ Closures : Velcro

SURGICAL DROP-OFF

APSD

TIE APRON



Our most basic apron. Easy to put on.

Ties are long enough to fit a large spectrum of sizes.

Chest pocket standard.

◆ Protection : .50mm front

◆ Closures : Ties

TIE APRON

APEA

VELCRO ADJUSTABLE



Crisscrossing back straps keep the shoulders from slipping off.
Velcro tabs make adjustability quick and easy.

Extra thick shoulder pads, for comfort and chest pocket are standard.

◆ Protection : .50mm front

◆ Closures : Velcro

VELCRO ADJUSTABLE

APVA

MATERNITY APRON



This apron features an adjustable Quick Release Buckle to secure the back panel and easy ties for the front panel allowing complete adjustability during pregnancy. Velcro release shoulders makes removal easy. Apron also includes an additional .50mm equivalent panel.

You can select either the Velcro attachment to the marsupial inner pouch or Velcro belt.

◆ Protection : .50mm front / 1.0mm(with added panel)
and .25 back

◆ Closures : Velcro

MATERNITY APRON

APMA

XGUARD GLOVE - RR2, RR3



FREEGUARD MODEL - 1,2 LEAD FREE, LATEX FREE, POWDER FREE



Our Soft-Touch finish creates a smooth interior reducing skin irritation. Anatomically correct curved fingers reduce hand fatigue.

Sizing range from 6 to 9 in half sizes for an unequalled comfortable fit. Attenuates up to 50% more direct beam radiation than other leading products. Thinner fingertips provide enhanced tactile sensitivity.

ATTENUATION

RR-2 : 0.012" / 0.30 mm				RR-3 : 0.024" / 0.60 mm			
Kvp	60	90	120	Kvp	60	90	120
Cuff	56%	44%	37%	Cuff	76%	63%	55%
Palm	57%	45%	38%	Palm	75%	62%	54%
Finger	57%	56%	38%	Finger	75%	62%	54%

ATTENUATION

MODEL 1 : 0.009" / 0.22 mm				MODEL 2 : 0.012" / 0.30 mm			
Kvp	60	90	120	Kvp	60	90	120
Cuff	51%	40%	34%	Cuff	62%	51%	43%
Palm	55%	44%	37%	Palm	63%	51%	43%
Finger	55%	44%	37%	Finger	61%	49%	42%

XGUARD SLEEVE



APRON SLEEVE



PROGUARD Radiation Reducing Sleeve, when used in conjunction with the PROGUARD Radiation Reducing Procedure Glove creates an attenuative barrier from your fingertips to your biceps from scatter radiation.

- ◆ Inner Nitrile coating - reducing proteins, antigens and irritation.
- ◆ Lightweight material reduces arm fatigue.
- ◆ Extreme flexibility does not restrict movement or dexterity.
- ◆ Available in three sizes for a comfortable fit.

This optional sleeve is the perfect solution for protecting the upper arm and shoulder. Right or left sides available.

Velcro tabs allow easy attachment or removal.

Permanent attachment is also available.

Features inner Velcro panels for custom girth adjustments.

Sleeve can also be retrofit to your existing apron.

ATTENUATION

XGUARD Sleeve	60 Kvp	80 Kvp	100 Kvp	120 Kvp	THICKNESS
	55%	45%	35%	31%	0.012" - 0.30mm

BREAST SHIELD	APSTOLE
SURGICAL DROP SHOULDER	APDRSH
THYROID COLLARS	APTCO / APTCS


BREAST SHIELD



SURGICAL DROP SHOULDER



THYROID COLLARS

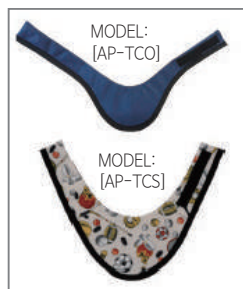

BREAST SHIELD	APSTOLE
SURGICAL DROP SHOULDER	APDRSH
THYROID COLLARS	APTCO / APTCS


Non Lead APRONS Color Swatches

NYLON


Unless specified otherwise, nylons come standard 210 denier.



RIPSTOP

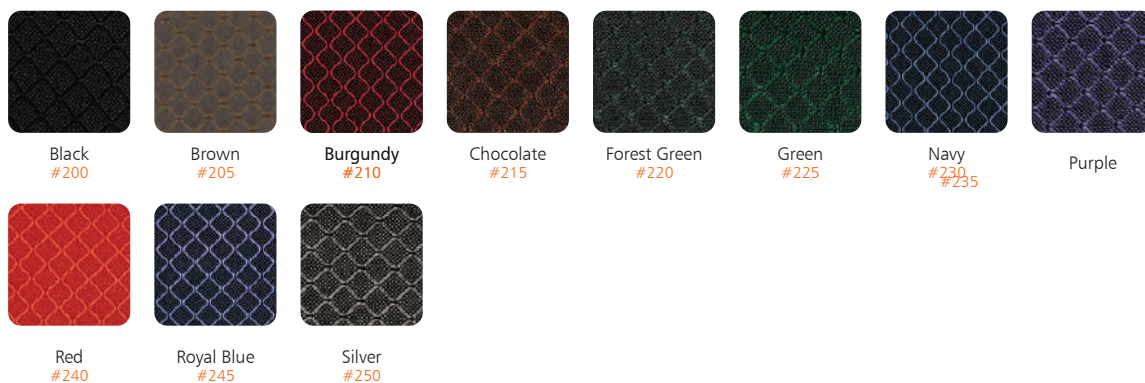

WIPE-AWAY VINYLs*

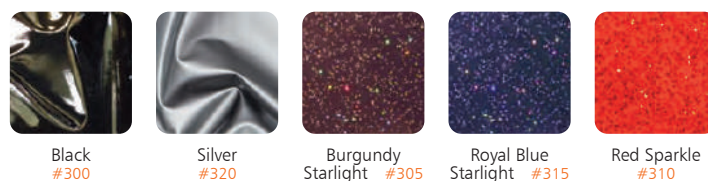

ANTI-MICROBIAL POLYS*


Featuring stain resistant and anti-microbial technology only available from Protech Medical. Our poly material will resist a wide variety of bacteria for several years.




PRINTS | FABRIC COLORS



PRINTS | POLYS

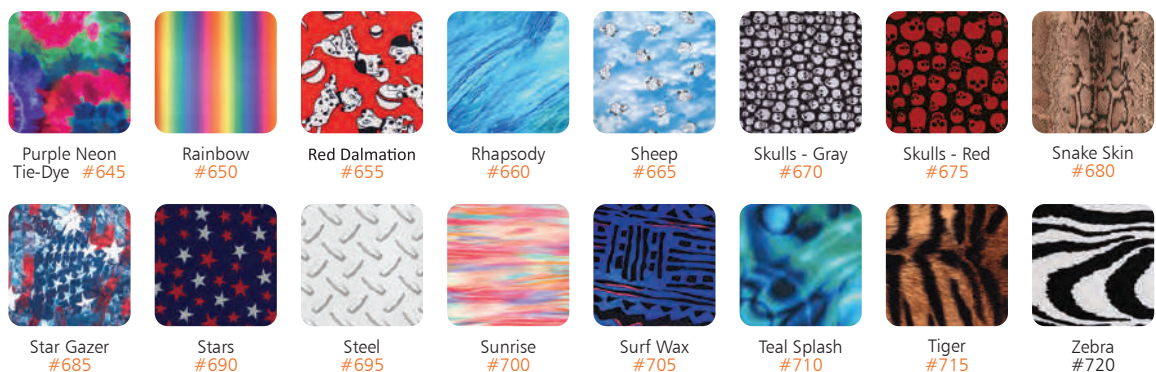

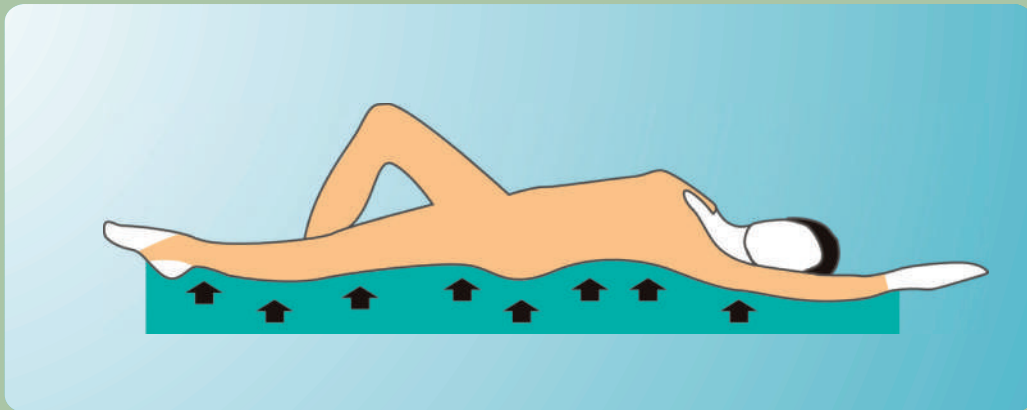

Sung Kwang Meditech Co.,Ltd

Information or ordering call

T) 82-31-855-7548 M) 82-10-4590-7548

F) 82-31-855-7549

E) skmeditech@skmeditech.com

Address) 1014-20, Bukhansan-ro, Jangheung-myeon,
Yangju-si, Gyeonggi-do, Korea Sung Kwang Meditech co.,Ltd



www.skmeditech.com